Service-Manual

PERFUSOR[®]-secura FT

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General Information

Please pay attention to the following information:

Servicing

This service manual is only intended for your information. The possession of this manual does not authorize servicing.

Servicing may only be performed if:

- you are trained on the respective unit by B. Braun,
- registered for update service,
- possess the necessary test equipments and tools and
- fulfill the following personal prerequisites (education, knowledge)

Technical Safety Inspections

B. Braun also recommends attending a training or at least perform the safety inspection according to the current manual version, because:

- the technical safety inspection demands observing the procedure information in this service manual,
- the service manual also represents a reference for measurements,
- the service program must be activated depending on the unit types. This can lead to dangerous unit conditions if handled improperly, besides a special service adapter is required to activate the service program.

Current Service Manual

This service manual represents the current status at the time of issue and is subjected to technical amendments, especially software. The change status of the service manual is indicated by an index number on the front page.

Update Service

The possession of this service manual does not automatically include an update service. Registration for update service by:

- attending a B. Braun Melsungen technical training or
- written order to the B. Braun sales department (against costs).

Exclusion of Liability

B. Braun Melsungen AG excludes all liabilities for personal, property or other injuries, which are caused by

- using a false service manual or a service manual which does not correspond to the current edition, during maintenance, repair and servicing of the unit,
- service technicians not registered for update service,
- service technicians not attending a technical training by B. Braun Melsungen AG for the respective unit.

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<i>Art. No.</i>	<i>Country</i>	<i>Keyboard (Language)</i>	<i>Type</i>	<i>integr. Power supply</i>	<i>Drawing-No.</i>
871 85210	Germany Austria Switzerland (G)	german	B	230V with Schuko plug	5163.1
871 85711	United Kingdom/ England	english	B (B+D)	240V with BSI-plug	5119.1
871 86817	France Switzerland (F)	french	B	230V with Schuko plug	5120.1
871 85319	Netherlands	dutch	B (SHE)	230V with Schuko plug	5121.1
871 86512	Italy Switzerland (I)	italian	B	230V with Schuko plug	5131.1
871 85613	Spain	spanish	B	230V with Schuko plug	5124.1
871 86719	Portugal	portuguese	B	230V with Schuko plug	5125.1
871 85819	Sweden	swedish	B	230V with Schuko plug	5126.1
871 85918	Finland	finnish	B	230V with Schuko plug	5127.1
871 86011	USA	english	B (B+D)	110V with Schuko plug	5128.1
871 86218	Denmark	danish	B	230V with Schuko plug	5130.1
871 86414	Ireland	english	B	230V with Schuko plug	5132.1
871 87019	Japan	english	B (Terumo)	100V with Schuko plug	5133.1
871 86110	Far east BMI	english	B	240V with Schuko plug	5134.1
871 85112	Near east	english	B	230V with Schuko plug	5136.1
871 86915	Turkey	turkish	B	230V with Schuko plug	5173.1
871 87814	Norway	norwegian	B	230V with Schuko plug	5172.1
871 86610	Far east Hongkong	english	B	200V with BSI-plug	5176.1
871 87411	Taiwan	english	B	120V with Schuko plug	5177.1

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The PERFUSOR® secura FT is especially suitable for longtime infusions of small volumes of medications at constant reproducible delivery rates.

A very fine setting of the delivery speed permits optimal adaptation to the required application when using the Original 50 ml PERFUSOR® syringe.

The PERFUSOR® secura FT has connection facilities for the staff call system.

The integrated accumulator in the PERFUSOR® secura FT permits mains independent operation.

In case of mains power failiure the PERFUSOR® secura FT automatically switches to accumulator operation.

The PERFUSOR® secura FT has been approved by the appropriate authorities and bears the TÜV/GS symbol. It complies with the Provision Regulating the Safety of Medicaltechnical Equipment (MedGV) and the German legislation covering technical equipment and safety (GSG).

The PERFUSOR® secura FT has been designed and built according to the regulations of the international Safety Standard IEC 601-1 (EN 60601-1).

The PERFUSOR® secura FT is equipped with a pre-alarm which gives an intermittent warning signal independent of the set delivery rate 3 min before the end of the infusion.

The PERFUSOR® secura FT has a data interface and is thus prepared for control through an external computer.

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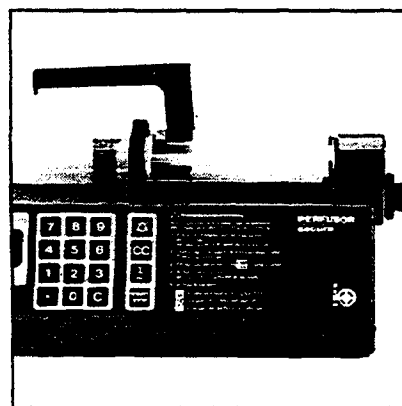
The PERFUSOR® securā FT has a pressure-path cut-out, i. e. as a general rule it is always switched off without any increase in pressure when the syringe is emptied. The pressure alarm only responds when there is an increase in the system. This increase can be seen from the spring-mounted pressure plate of the rear syringe holder.

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The spring-mounted pressure plate also provides a check on whether a syringe is fitted in the rear syringe holder. The user shall check that the syringe is positioned properly.

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The syringe is safely fixed by the safety clamp.

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After unpacking the unit check for completeness and damage.

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Important!

If the unit has become damaged it shall not be put into operation. Please inform the responsible service station.

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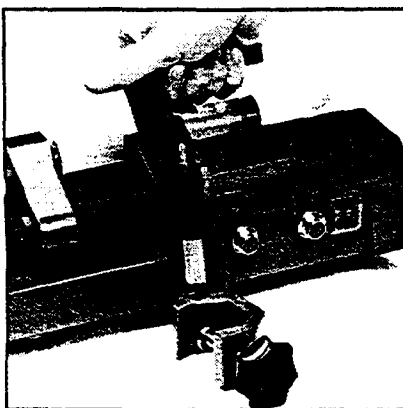
The existing mains voltage shall correspond with the mains voltage given on the main label of the power supply.

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Connection to internal staff call system (if given):
Insert special lead (offered as optional accessory) in socket and fix properly.

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The terminal block with hexagonal socket permits fastening to all common mounting systems, i. e. infusion poles, wall rail systems (BRAUNOSTAT®), etc.

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Fundamental, the mains-power input at the device shall be protected against wetness.

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Caution: Follow the instructions für use (operator's manual).

The PERFUSOR® secura FT shall only be operated in medically used rooms where the installation is in compliance to the applicable standards respectively to National Requirements (e. g. IEC-publications).

The PERFUSOR® secura FT shall not be operated in presence of flammable anaesthetics.

The PERFUSOR® secura FT shall only be operated with "Original-PERFUSOR®-Disposables" or approved disposables.

The "Original-PERFUSOR®-Disposables" are intended for single use. Syringes have to be changed after emptying, at least however after 24 h.

The functional safety of the PERFUSOR® secura FT cannot be guaranteed if unapproved accessories are used.

All additional accessories connected to the "Original-PERFUSOR®-Disposables" (e. g. injection sites, 3-way stop cocks, additional infusion lines etc.) shall meet the technical data for the recommended infusion-administration-sets for the power infusion pumps.

Attention: Use only needles, cannulae or catheters which are suitable for the specific treatment. Caution: Avoid bending of the infusions line.

When you combine different apparatus and infusion-administration-sets, a reciprocal influence cannot be excluded

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Operating PERFUSOR® secura FT in the range of occlusion pressure can influence the accuracy of the delivery rate.

Before starting any operation the user check has to be done.

The PERFUSOR® secura FT shall only be operated by trained persons. The PERFUSOR® secura FT shall be operated under periodic control of physicians or clinical staff.

Caution: In case of mounting e. g. on an infusion pole check for the stability.

The connection to the patient may only exist, if the unit is switched on.

Bolus volume max. 2 ml (at 40°C and 1,2 bar) by using an Original-PERFUSOR®-syringe and an Original-PERFUSOR®-administration set (872 296/0).

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Only use admissible disposable articles!

Occlusion pressure: max. 120 kPa (900 mm Hg)

1. To switch on, turn on ON/OFF switch at back of **PERFUSOR®** secura FT (syringe type printed on the adapter must appear in the display)

2. Release rotary knob, fit syringe and lock rotary knob

3. Set delivery rate, press START/STOP key, to start: compare input-value with display-value

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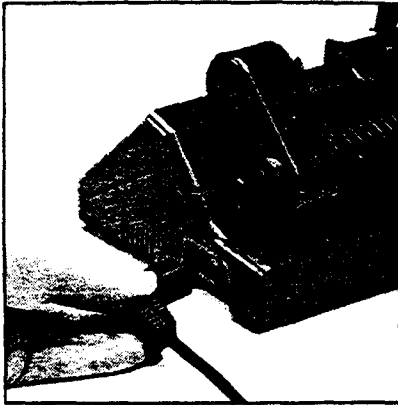
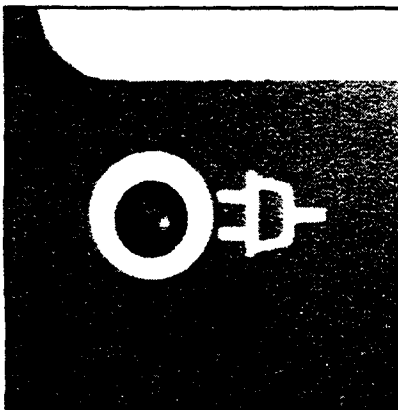
Before starting treatment, user check has to be done (see page 23.)

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Mains connection:
Connect plug with socket (18).
Alternatively the PERFUSOR® secura FT can be operated by an external power supply.
In this case you have to connect the plug of this unit with socket (14). (Screwed connection!).

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The green LED (24) signals connection with the mains and at the same time the integrated accumulator will be charged, independent of operating status of the unit.

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Bring syringe holder into starting position:
Release the resetting knob by pulling.

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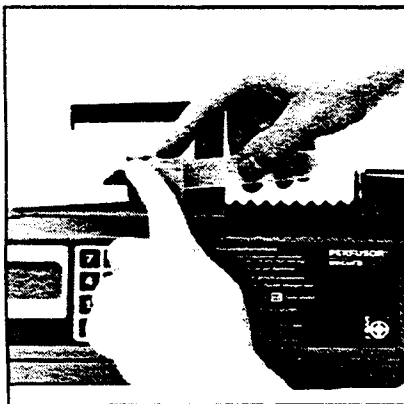
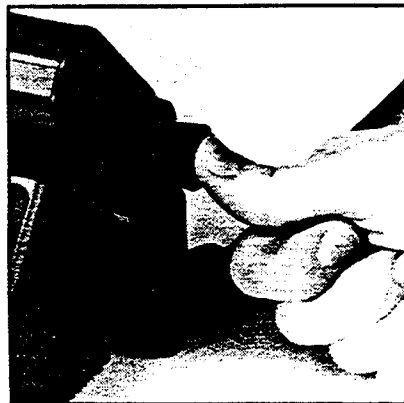
Push syringe holder in starting position. Pay attention that you do not hamper the motion of the resetting knob. Fine setting is possible by turning the resetting knob in both directions.

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Fit syringe: turn the safety clamp and press the syringe together with the cylinder grip plates in the front holder and the pressure plate of the piston in the rear syringe holder. Put the safety clamp back into the initial position.

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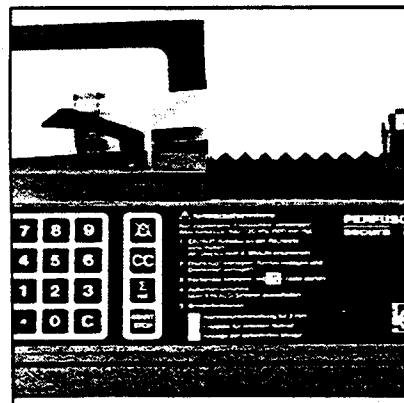
To vent infusion lines: By turning the reset knob in clockwise direction the infusion line is vented and at the same time any play in the drive system is eliminated. Lock reset knob by pressing down.

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Switch on PERFUSOR® secura FT: Actuate on/off switch at the back.

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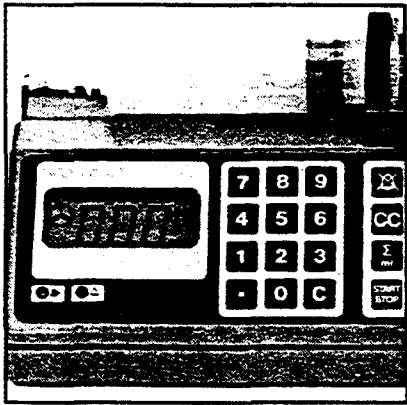
Important:
Press the syringe up against the stop in the holder.
Fig.: Unproperly positioned syringe.

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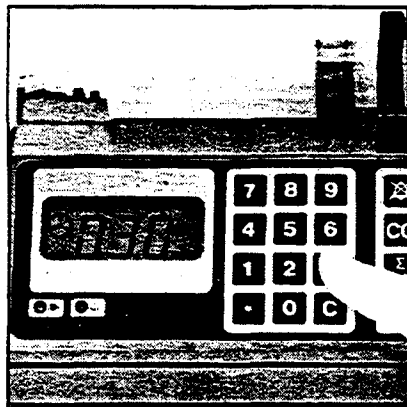
Self check:

After being switched on the system automatically checks itself. All display elements and the audible alarm will be briefly activated (pay attention to code 111, 222, 555). After the self check has been completed, syringe type printed on the adapter must appear in the display.

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To set delivery rate:

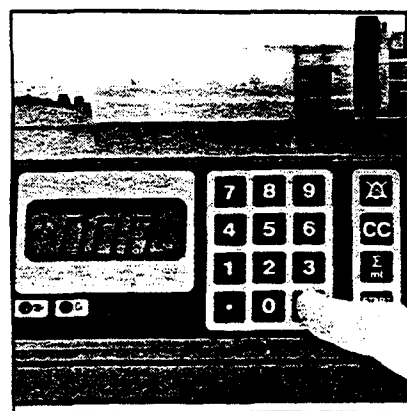
Enter required delivery rate between 0.1 ... 99.9 ml/h in the digit field.
(i. e: 0 · 5 = 0,5 ml/h,
3 = 3,0 ml/h)

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Display:

The following will appear in the display, e. g. 0.5 or 3.0 ml/h respectively. Check that the display value is in accordance with input value. Operate the unit only if compliance is given.

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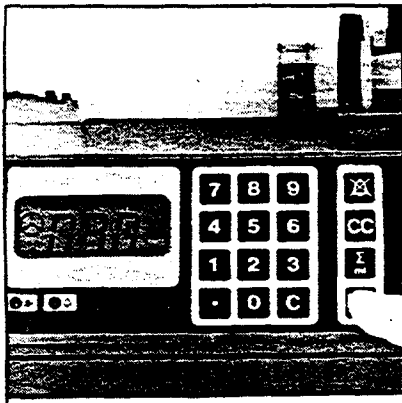
To correct entry:

By pressing the "C" key the display is set to 00.0. Now a new delivery rate can be entered.

Caution: Connect the patient only, if the syringe is put correctly in the PERFUSOR® secūra FT and the tubing is ventilated.

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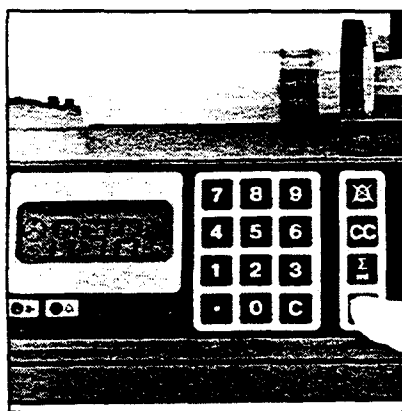
To start PERFUSOR® secūra FT:

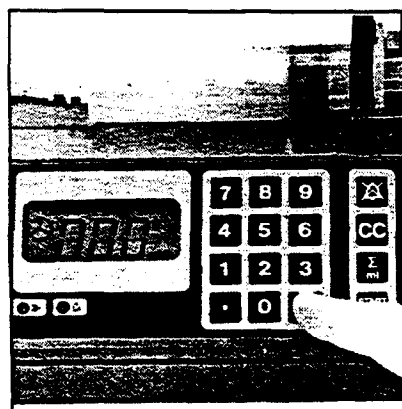
After controlling the correct delivery rate the unit is to be started by pressing the Start/Stop button.

The operating indicator in the LCD display starts to turn and shows the operational state.

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Change the delivery rate: Interruption of the infusion by pressing the start/stop key. Changes may only be carried out when the unit is stopped. By pressing the "C" key the display is set to 00.0. Now a new delivery rate can be entered. Then press start/stop button again.

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Comment:

The changing of the delivery rate is not possible without pressing the "C" key.

Pre-alarm:

3 min before the end of the infusion an intermittent warning tone will be given. The running control will continue to turn and the syringe symbol in the LCD display flashes.

End of infusion:

At the end of the infusion the red alarm LED is illuminated, the syringe logo is flashing and the audible alarm signal is given.

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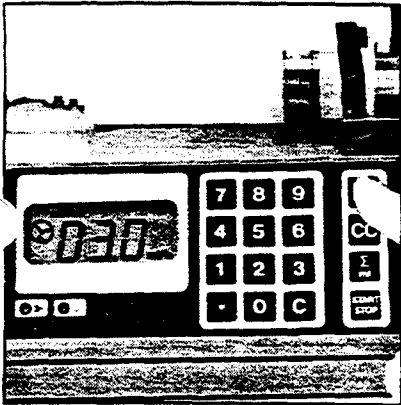
Restart:

After removing the emptied syringe the unit is started again as described from "Bringing syringe holder into starting position".

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The audible alarm can be interrupted for 2 min by actuating the key .

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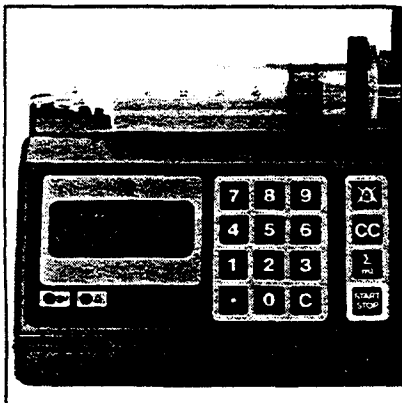
To switch-off the unit:

Press the ON/OFF button at the back of the unit.

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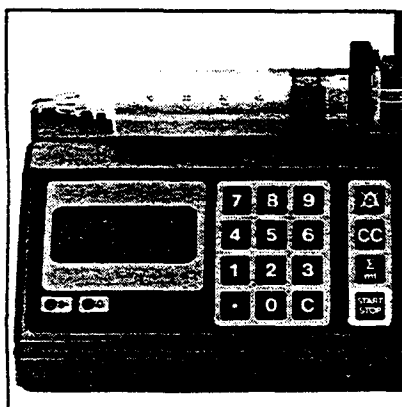
Accumulator operation:

The accumulator operation is shown by the accumulator symbol on the display (19).

Putting into operation follows as described under mains operation.

In the case of a mains power failure the PERFUSOR® secura FT automatically switches to accumulator operation.

During accumulator operation the display is not illuminated.

About 3 min before the accumulator is discharged an accumulator pre-alarm will be generated as follows:

- Flashing accumulator symbol
- Audible signal

At the end of the low battery operation alarm is activated:

- Battery symbol flashes
- Red alarm LED
- Staff call
- Audible alarm

The unit alarm signal is automatically given when the accumulator is discharged.

The unit shall be switched off immediately or operated from the mains power supply.

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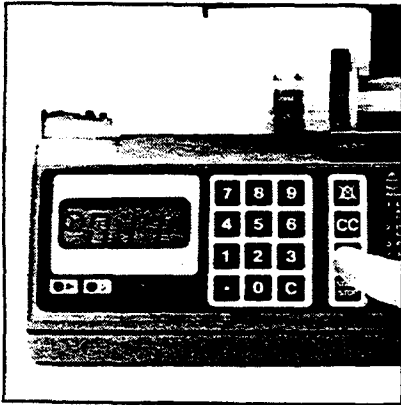
Instructions for accumulator operation:

- When used by mains green control Indicator (24) comes on.
- Charging is given independent of the operation-mode.
- At the first operation procedures and longer disconnection from mains (about 2 month) the accumulator shall be charged.
- The cells are fully charged after 16 hours. Because of an integrated Charging-Timer, the charging period will not have a negative influence to the accumulator.
- With fully charged accumulator the PERFUSOR® *secura* FT can be run approx. 4 h at maximum delivery rate.
- The charging should possibly be done between +5°C and +35°C.
- The accumulator will last longest if is fully charged from time to time.
- Under normal use the capacity of the cells will slowly weaken. After approx. 2 years it will only work at half capacity.

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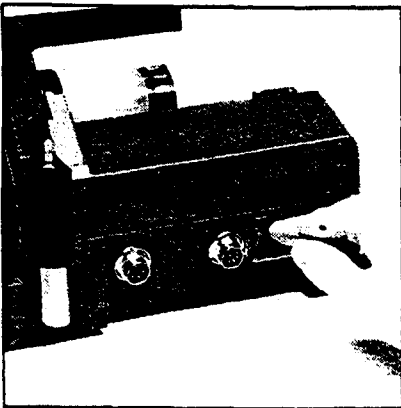
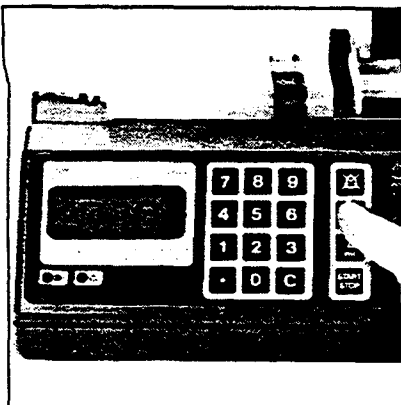
Take out the delivered volume:

The delivered volume after switching on will be displayed in ml for approx. 2 s when pressing the key Σ ml. If the "C" key is pressed during this display the total sum will be set to 00.0. Cancelling will only be possible if the unit is stopped.

When switching on the PERFUSOR® securā FT the volume display will be set to 00.0 so that the volume can be added when a syringe is changed.

Until 99.9 ml the delivered volume is shown in 0,1 ml-steps. From 100 ml to 999 ml the addition in the display will be done in full ml-steps.

Display illumination:
Only during mains operation.

Interface operation:
The CC key is used for the interface. Interface operation is only admissible with approved supplementary equipment.

The operating sequence is to be seen on the operation instructions for this supplementary equipment. The CC key is only activated with connected supplementary equipment.

Do not touch the contacts. Static charges could damage the electronic circuits.

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1:9 = 1 s akustischer Alarm, 9 s Pause

Faults

Warning	Fault	Remedy
Red LED comes on Oper. indicator at standstill Syringes flashes Audible alarm 1:4 Staff call	Pressure in system too high	Eliminate occlusion Possibly catheter change necessary Restart
	Syringe empty	Change syringe Restart or end infusion Switch off unit
	Syringe fitted wrongly or not at all	Fit Syringe correctly Restart
	Reset knob still released	Engage reset knob Restart
Red LED comes on briefly Oper. indicator at standstill Audible alarm	Delivery rate not set Display 00.0! C-key not activated before changing delivery rate	Enter delivery rate Restart Press C-key, enter rate Restart
	Unit has not started	Enter delivery rate Restart Start unit
Syringe flashes Audible signal 1:9 operating display rotates	Pre-alarm 3 min before end of infusion	No malfunction since unit is running normally and giving pre-alarm signal
Accumulator flashes Audible signal 1:9 operating indicator rotates	Accumulator pre-alarm	Continue to run unit from mains
Battery flashes Audible alarm 1:4 Oper. indicator at standstill Red LED flashes Staff call	Accumulator discharged Accumulator pre-alarm signal has been overlocked	Switch off immediately or operate from mains
Red LED comes on Oper. indicator at standstill Continuous tone Staff call	Accumulator empty	Switch off immediately change accumulator
Red LED comes on Oper. indicator at standstill All symbols signal Continuous tone Staff call	Technical failure	Send unit for repair
Red LED comes on, operating indicator shows 9 spokes Continuous tone	During selfcheck a keyboard button was pressed	Switch off the unit and start again

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Responsibility of the manufacturer

The manufacturer, assembler, installer or importer considers himself only responsible for the effects on safety, reliability and performance of the equipment if:

- assembly operations, extensions, readjustments, modifications or repairs are carried out by persons authorized by him and
- the electrical installations of the relevant room complies with the VDE 0100/VDE 0107/IEC publications.
- the equipment is used in accordance with the instructions for use.

Note: Check compliance with national requirements!

Warranty:

B. BRAUN MELSUNGEN AG provides a warranty period of 24 months from the date of supply. The guarantee covers the installation and the provision for replacement parts, provided that these can be shown to relate to construction, fabrication or material defects. The guarantee becomes void if the owner or a third party undertakes repairs or alterations to the device.

Excluded from the guarantee are faults arising from mishandling of the device and unauthorised usage for non-approved applications.

Safety checks should be conducted at 24 monthly intervals and should be documented in the appropriate device manual.

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Regular servicing

A complete service handbook is available on request but only in conjunction with technical training.

Servicing is recommended every 2 years.

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For the proper condition and the safety of function and in conjunction with the instructions for use

- check of cleanliness, completeness
- check of damage
- check of power supply (mains lead, connector)
- check of the staff call (if connected)
- check the spindle
- switch on the unit self check (main switch, alarm sound, alarm indicator)
- pay attention to code 111, 222, 555
- check the accessories
- getting started according to the instructions for use

Cleaning and Disinfection

The PERFUSOR® secura FT can be cleaned with common household detergent.

For disinfection, only surface-acting reagents shall be used.

The PERFUSOR® secura FT can be **disinfected with a suitable cloth**, when the cloth has been soaked with a disinfectant before.

The device should be left to stand for at least 1 min after treatment to allow for residual vapour to evaporate.

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Protection class: II, Type: CF

RFI:

B (according to DIN VDE-0871 and DIN VDE 0875, respectively EEC-Directive 82/499 and CISPR 11)

Moisture protection:

drip-proof

Duration of operation:

100 % (cont)

Environmental conditions (without condensation)

a. operation

relative humidity:

30 % ... 75 %

temperature:

+ 5 °C ... + 40 °C

– atm. pressure:

500 mbar ... 1060 mbar

b. Transport and storage

– relative humidity:

10 % ... 90 %

– temperature:

– 25 °C ... + 55 °C

Accumulator type:

NiCad, Ord.-Nr. 3317024/0 (rechargeable)

Accumulator:

> 4 h at max. delivery rate

Recharge time:

> 16 h

Pre-alarm:

3 min before end of infusion

Deviation of preselected delivery rate (including the

± 2 %, duration of measurement > 1 h, delivered volume > 2 ml

disposables recommended by the manufacturer:

Automatic alarm activated in case of dosage failure:

At malfunction of the unit the pump will stop in case of dosage failure of max. 0.2 µl.

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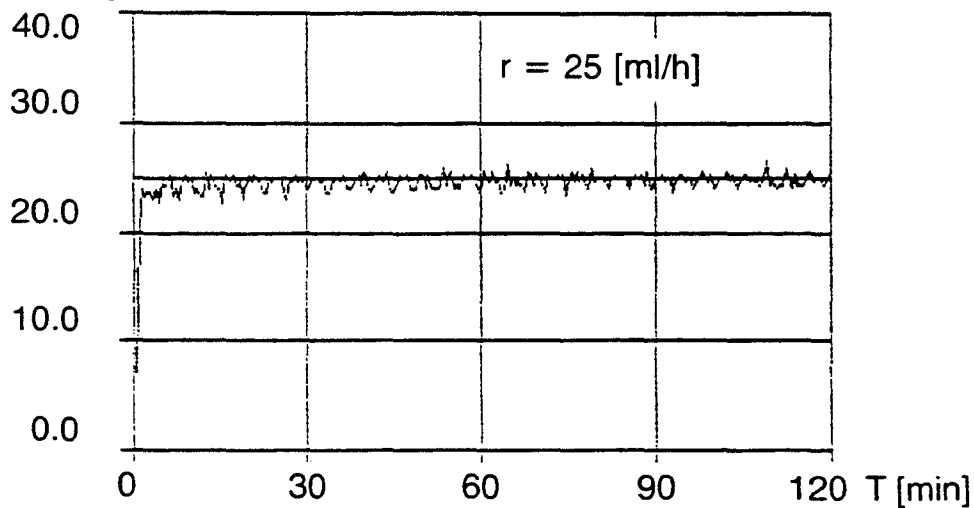
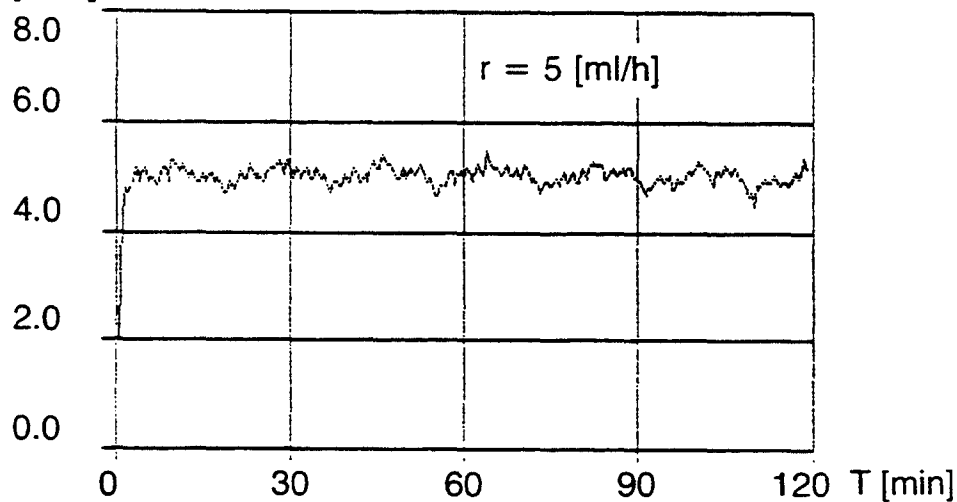
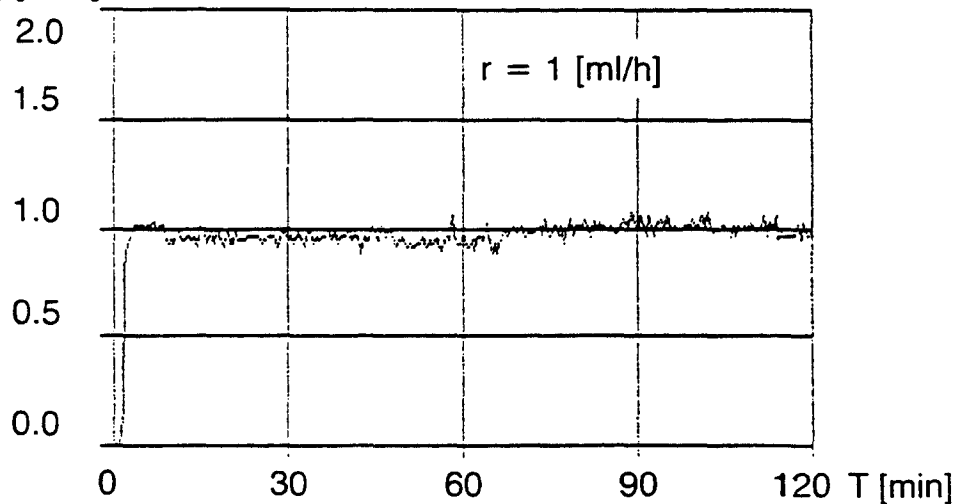
Trumpet curves (Measured values from 2nd hour)

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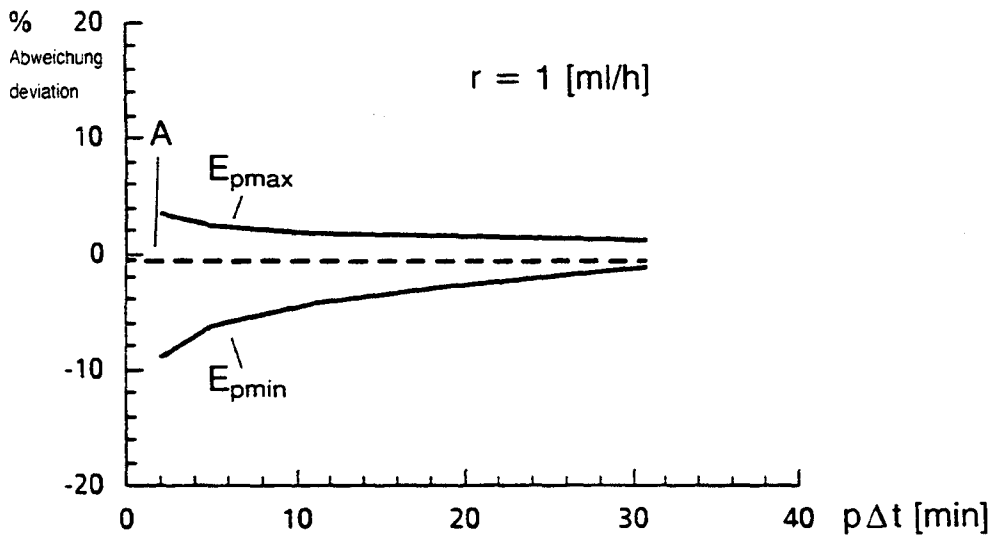
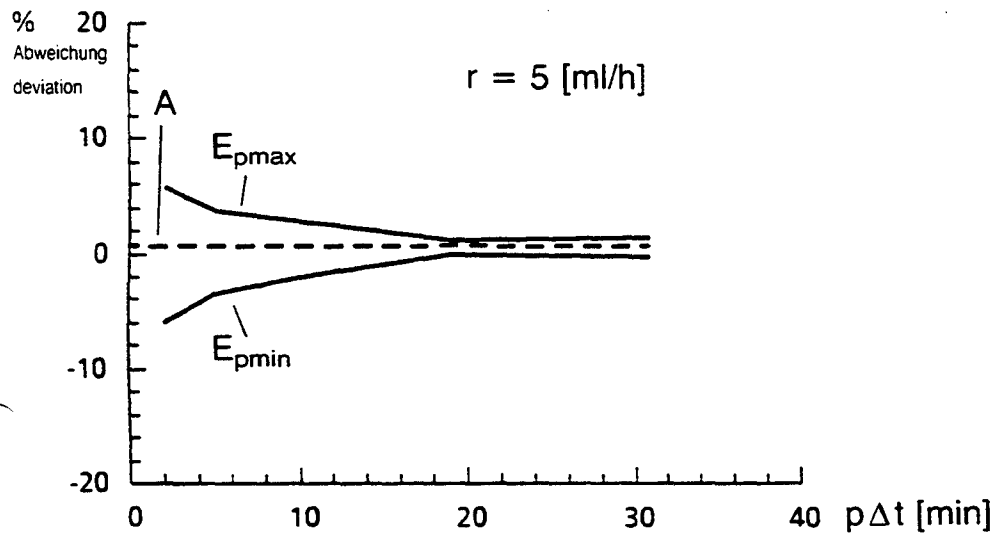
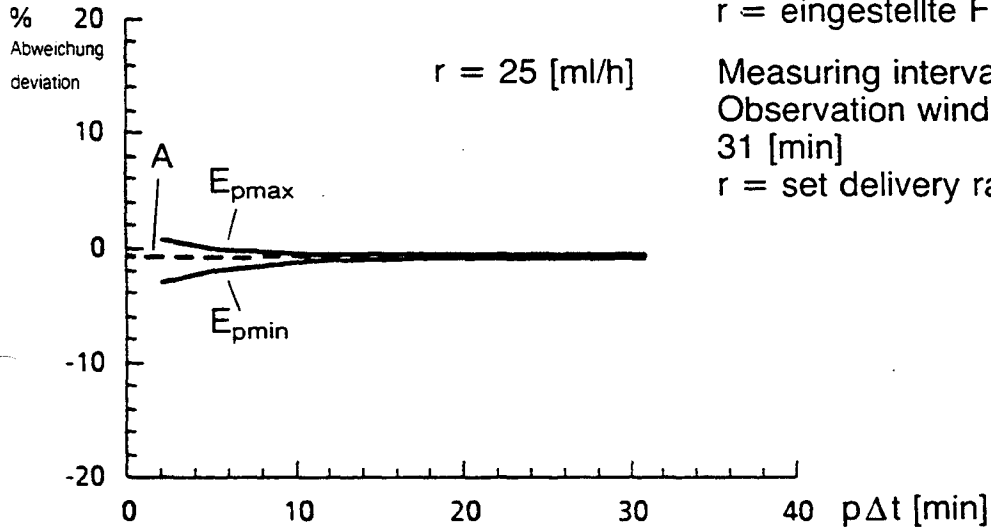
Formulae

T = Test period where $T = t_k - t_1$ [min] and k = maximum number of interrogation points

k = number of interrogation points

r = set delivery rate [ml/h]

i = index of interrogation points where $i = 1$ to k

Δt = interval between successive interrogation points [min]

t_i = interrogation time [min] at interrogation point i

p = observation window index [where $p = 4, 10, 22, 38$ and 62]

$p\Delta t$ = observation window where $t_{p,i} = t_{i+p} - t_i$ [min]

w_i = measured weights [g] corrected for evaporative loss, at interrogation point i

d = density of water = 0.998 g/ml at 20°C

Q_i = flow [ml/h] measured between successive interrogation points
$$Q_i = \frac{60(w_i - w_{i-1})}{d \cdot \Delta t}$$

$w_{p,i}$ = weight [g] delivered in observation period $t_{p,i}$
 $w_{p,i} = w_{i+p} - w_i$

W = total weight delivered [g] in test period T corrected for evaporative loss
 $W = w_k - w_0$

Q = mean flow [ml/h] in test period T
$$Q = \frac{60W}{d \cdot T}$$

$Q_{p,i}$ = mean flow (ml/h) in observation period $t_{p,i}$
$$Q_{p,i} = \frac{60w_{p,i}}{d \cdot t_{p,i}} = \frac{120w_{p,i}}{d \cdot p \cdot \Delta t}$$

Q_{pmax} = maximum $Q_{p,i}$ [ml/h] in any one observation period $t_{p,i}$ for each value of p

Q_{pmin} = minimum $Q_{p,i}$ [ml/h] in any one observation period $t_{p,i}$ for each value of p

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$$A = \frac{Q - r}{r} \cdot 100\%$$

E_{pmax} = largest upwards variation in measured flow in any one observation period $t_{p,i}$ for each value of p expressed as a % error on the set delivery rate

$$E_{pmax} = \frac{Q_{pmax} - r}{r} \cdot 100\%$$

E_{pmin} = largest downwards variation in measured flow rate in any one observation period $t_{p,i}$ for each value of p expressed as a % error on the set delivery rate

$$E_{pmin} = \frac{Q_{pmin} - r}{r} \cdot 100\%$$

A = overall % error in flow measured over the test period T

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Comment: Start-up-and Trumpet-curves

The graphics shall give a representation to the medical user, how exact and regular an infusion system works.

It must be taken into account that the exactness of the set delivery rate mainly depends on the used disposables (syringes, infusion lines).

If disposables are used which are not called in the „order data“, the deviations can vary significantly.

● Start-up-curves:

These curves show the typical start-up of the infusion system at different delivery rates. The curves shows after which test-period (T) the set delivery rate is reached.

● Trumpet curves:

The trumpet curves show the influence of flow-swayings to the exactness in dependency of the time.

The influence of shorttime flow-swayings to the delivery reaction decreases when delivery rates increase. At critical applications (e. g. by using medicaments with low half life times) we recommend delivery rates > 1 ml/h.

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Symbol explanation

-  Drip proof
-  Attention consult accompanying documents
-  Protection class II
-  CF-application
(cardiac floating)
-  Interface
-  Staff call
-  ON (only for one part of the unit)
-  OFF (only for one part of the unit)
-  Potential equalization

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Original-PERFUSOR®-Leitung PES 1 × 2 (mm), 150 cm lang,

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Recommended Disposables

Original-PERFUSOR®-Syringe 50 ml with cannula	Cod.-No. 872881/0
Original-PERFUSOR®-Syringe 50 ml with cannula and 15 µm-filter, light-protected	Cod.-No. 872882/8
Original-PERFUSOR®-Syringe 50 ml without cannula	Cod.-No. 872884/4
Original-PERFUSOR®-Syringe 50 ml with cannula and 15 µm-filter	Cod.-No. 872885/2
Original-PERFUSOR®-Tubing L like Cod.-No. 872296/0 however 200 cm long, pressure proofed	Cod.-No. 872286/2
Original-PERFUSOR®-Tubing S 1.5 × 2.7 (mm), 150 cm long, PVC light-protected with Luer-Lock-connectors, pressure proofed	Cod.-No. 872291/9
Original-PERFUSOR®-Tubing PE 1 × 2 (mm), 150 cm long, Polyethylene with Luer-Lock-connectors, pressure proofed	Cod.-No. 872293/5
Original-PERFUSOR®-Tubing N PVC, 1.5 × 2.7 (mm), 150 cm long, with Luer-Lock-connectors, pressure proofed	Cod.-No. 872296/0
Original-PERFUSOR®-Tubing M PVC, 1.5 × 2.7 (mm), 150 cm long, with at patient's end, loose lock nut, pressure proofed	Cod.-No. 872299/4
Original-PERFUSOR®-Tubing PES 1 × 2 (mm), 150 cm long, PE light- protected with Luer-Lock-connectors, pressure proofed	Cod.-No. 872301/0
Original-PERFUSOR®-Tubing MR 0.9 × 2.0 (mm), 75 cm long, PVC, Luer-Lock-connectors, pressure proofed	Cod.-No. 872287/0
Original-PERFUSOR®-Tubing MK 0.6 × 1.6 (mm), 75 cm long, PVC, with cannula, Luer-Lock-connectors, pressure proofed	Cod.-No. 872288/9
Original-PERFUSOR®-Tubing 1.5 × 2.7 (mm), 200 cm long, PVC, with filter 0.22 micro, Luer-Lock-connectors, pressure proofed	Cod.-No. 872300/1

Recommended accessories:

Connection lead for staff call	Cod.-No. 870160/1
Connection lead for potential equalization	Cod.-No. 870161/0
Connection lead for an ambulance car (12 V)	Cod.-No. 871160/7
Dianet sc for 230 V ~ 50/60 Hz with "Schuko"-plug	Cod.-No. 871932/2
Infucommand, Microprocessor based controlling unit for treatment of patient controlled analgesia (PCA)	Cod.-No. 871782/6

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Melsungen, 01. 10. 90

Ort, Datum

Certification of Manufacturer/Importer

Herewith we are declaring that the

Spritzen-Infusionspumpe, PERFUSOR® secura FT

(Device, Type, Model)

the interference suppression meets the requirements of
"VfG 1046/1984"

The marketing of this device was announced to the German Bundespost. The German Bundespost is authorized to control the compliance with these requirements.

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This maintenance reference must not be copied, partly or
whole, nor made accessible to third parties without our
consent, and remains the property of our company.

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Faults: PTA 004 Software

WARNING	FAULT	REMEDY
Red LED comes on Operating indicator at stillstand Syringe flashes 1:4 Audible alarm Staff call	Pressure in system too high Syringe empty Syringe fitted wrongly or not at all Reset knob still released	Eliminate occlusion Possible catheter change necessary Restart Change syringe Restart or end in- fusion Switch off unit Fit syringe correctly Restart Engage reset knob Restart
Red LED comes on briefly Operating indicator at stillstand Audible alarm	Delivery rate not set Display 00.0 ! Before rate change "C" Key not actuated	Enter delivery rate Restart
Red LED comes on briefly Operating indicator at stillstand Audible alarm Staff call	Unit not started	Enter delivery rate Restart Start unit
Syringe flashes 1:9 Audible alarm Operating indicator rotates	Prealarm 3 min. before end of infusion	System is not faulty since unit is running normally and giving prealarm signal
Accumulator logo flashes 1:9 Audible alarm Operating indicator rotates	Accumulator prealarm	Continue to run unit from mains
Battery flashes 1:4 Audible alarm Operating indicator at stillstand Red LED flashes Staff call	Accumulator empty Accumulator prealarm signal has been over- locked	Switch off immediately or operate from mains
Red LED comes on Operating indicator at stillstand Continuous tone Staff call	Accumulator empty	Switch off immediately and charge accumulator
Red LED comes on Continuous tone Operating indicator shows 9 spokes Staff call	Unit has broken down	Send unit for repair
Red LED comes on Continuous tone Staff call	During selfcheck at keyboard button was pressed	Switch off the unit and start again

1:4 = 1 sec. audible alarm
4 sec. break
1:9 = 1 sec. audible alarm
9 sec. break

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Special functions with test adapter

Perfusor-secura FT
Software Version PTA 004

Software version display

During the switching-on test the software version will be shown in the display instead of 111/222/555; it appears as A04 in the display.

Programming of CC number (Ident. No.)

The rate value shown will be taken over as CC number; preselect rate and actuate CC key. The selected figure appears in the rate display for 2 seconds. Switch off unit. The CC number is programmed (setting range: 01 to 99).

Calling up the working hours

After switching on do not preselect any rate value (rate = 0) and actuate START/ STOP key. The working hours will be displayed for two seconds.
Rate value x 100 = working hours.

Calling up accumulator threshold

By actuating the alarm suppression key the accumulator threshold will be shown for 5 seconds in the display (b00 to b15).

Adjusting the accumulator threshold

The threshold value will be raised by 1 by actuating the comma key when calling up the accumulator threshold value. (b00...b15 overflowing to 0)

Automatic compensation of accumulator threshold

Set accumulator threshold manually to 0. Then charge the unit for 20 hours and discharge for 6 hours in accumulator operation. The accumulator threshold is now balanced out automatically, call up accumulator threshold value once again and check whether the accumulator threshold has been raised to a value approx. 0; if the accumulator threshold value has not been raised then the accumulator is defective or the balancing out instructions were not carried out correctly.

Important!

- This balancing out operation has to be carried out after changing an accumulator.
- This balancing out operation has to be carried out after changing an microprocessor circuit board.

Alternatively the value can also be taken over and given in from the first circuit board.

The rules of CMOS Technic have to be noted when working on circuit boards!

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Test points when changing the microprocessor circuit board.

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The following voltages have to be measured to avoid follow-up damage caused by a defective analogue circuit board.

Operate unit in mains operation without accumulator and without testing plug!

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Reference point is the "GND" soldering eyelet.

Stecker P6 Pin No.	Bezeichnung Designation	Meßwert Measuring value	
3a	LED - GRUEN	1,8-2,5V	nur messen, falls Netz-LED defekt measure only, if mains-LED is defective

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For further measurements pull off plug P6 (remove housing upper part)

2b	LED - I	ca. 16mA approx.	Ersatzweise kann im Normalbetrieb 0,5V-0,7V über R37 gemessen werden In normal operation 0.5V-0.7V it can be measured as compensation over R37
2a	6,2V	≧ 6,5V	
8a	REL-RUECK	≧ 5,5V	
8b	AK AL	≧ 5,5V	
19a	5VK	≧ 5,5V	
17a	SLEEP	≧ 6,5V	

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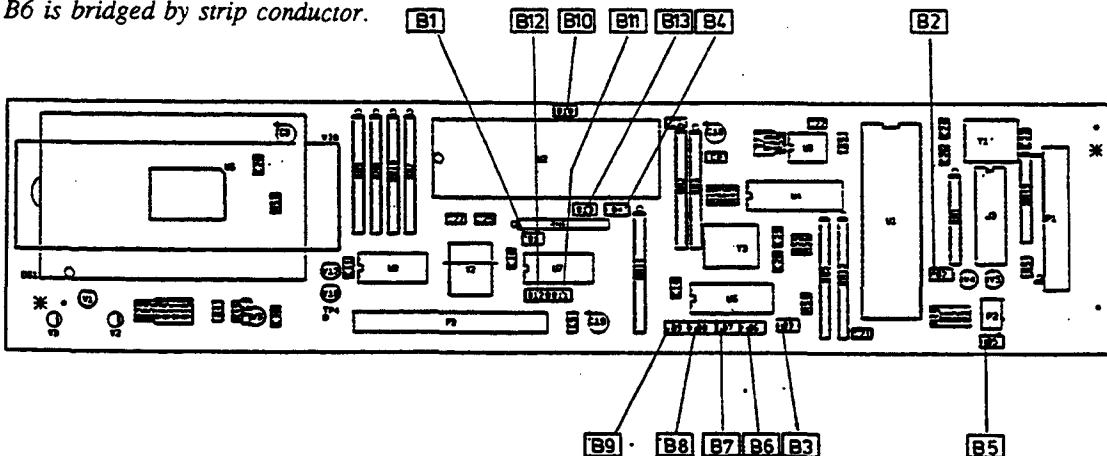
When exchanging the Microprocessor-circuit board please note that the soldered bridges B1-B5 and B7-B13 are not closed!

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To replace and fit bellow

- 1.) Carefully insert the light barrier (2i) in the guide tube
- 2.) Push the bellow (2a) over the guide tube and mount on drive housing

ATTENTION: Push bellow chank of a groove over the guide tube in direction of arrow!

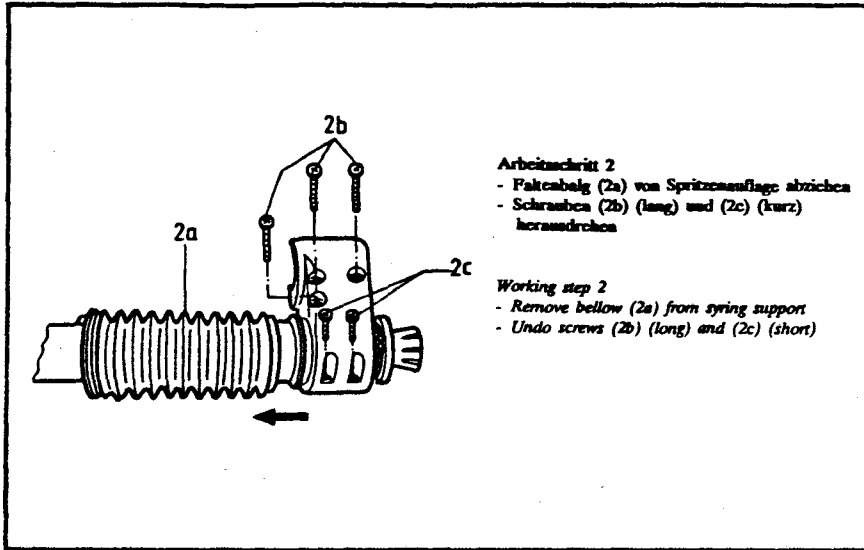
- Carefully press the legs of LB-carrier (2i) together and connect with the light barrier (2i)
 - Fix the LB-carrier (2i) and guide tube in the syringe support lower part
 - Fit unlocking facility (2h) on ball spindle
- 3.) Fit button switch leg (2g) and cylindrical pin (2f)

NOTE: Button switch leg (2g) must engage in guide slot of unlocking facility (2h)!!

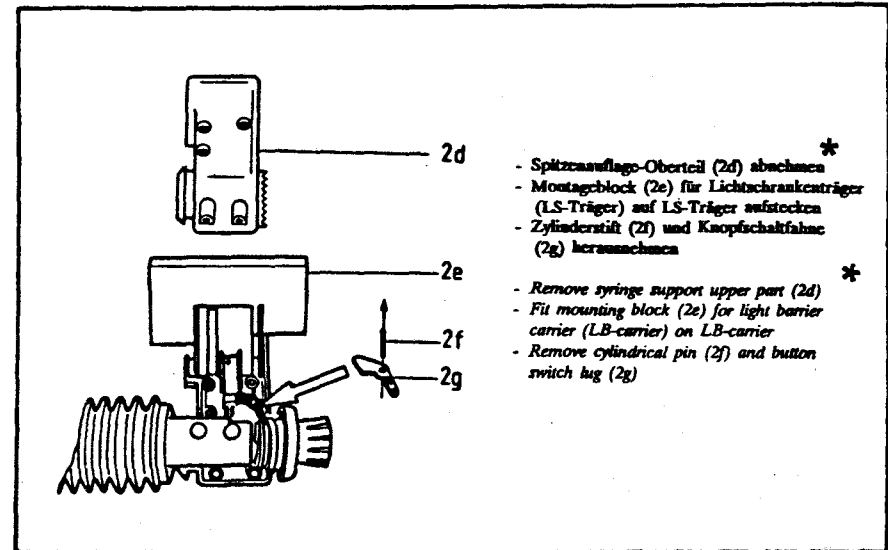
- Remove mounting block (2a) and fit the syringe support upper part (2d)
- Tighten screws (2c) (short) and (2b) (long)
- Fit bellow (2a) on syringe support
- Fit locking washer (1c) and do up nut (1b), fit cover cap

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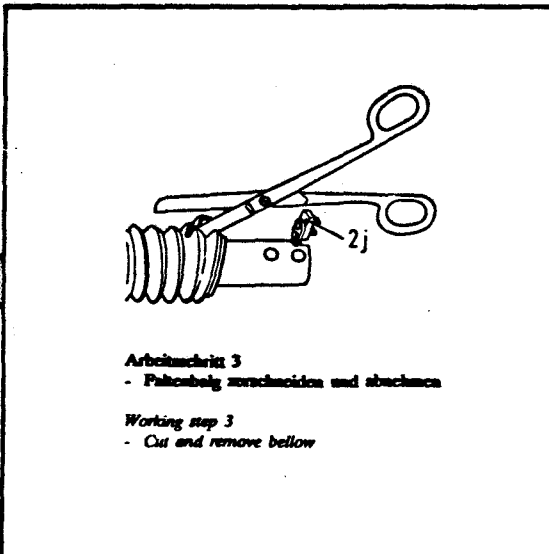
Repair instruction drive "subdomain A"

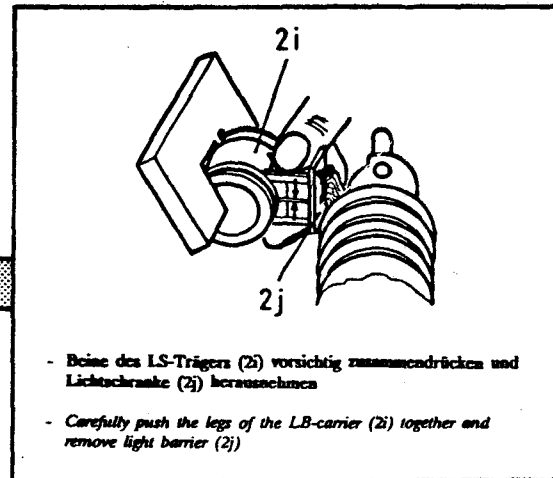
- Arbeitsschritt 2**
- Faltenbalg (2a) von Spritzenanflage abziehen
 - Schrauben (2b) (lang) und (2c) (kurz) herausdrehen
- Working step 2**
- Remove bellow (2a) from syringe support
 - Undo screws (2b) (long) and (2c) (short)

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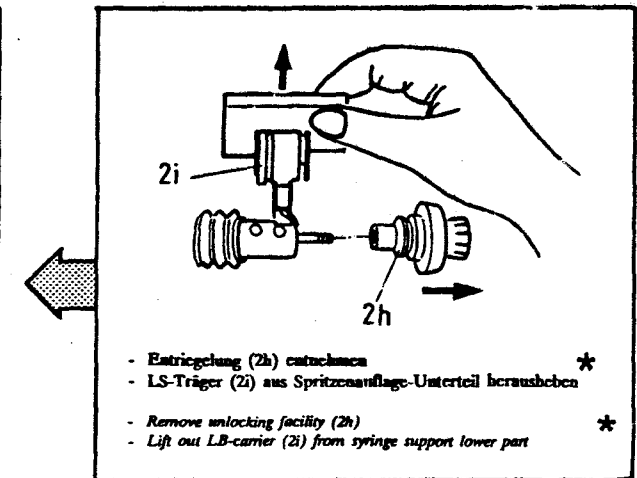
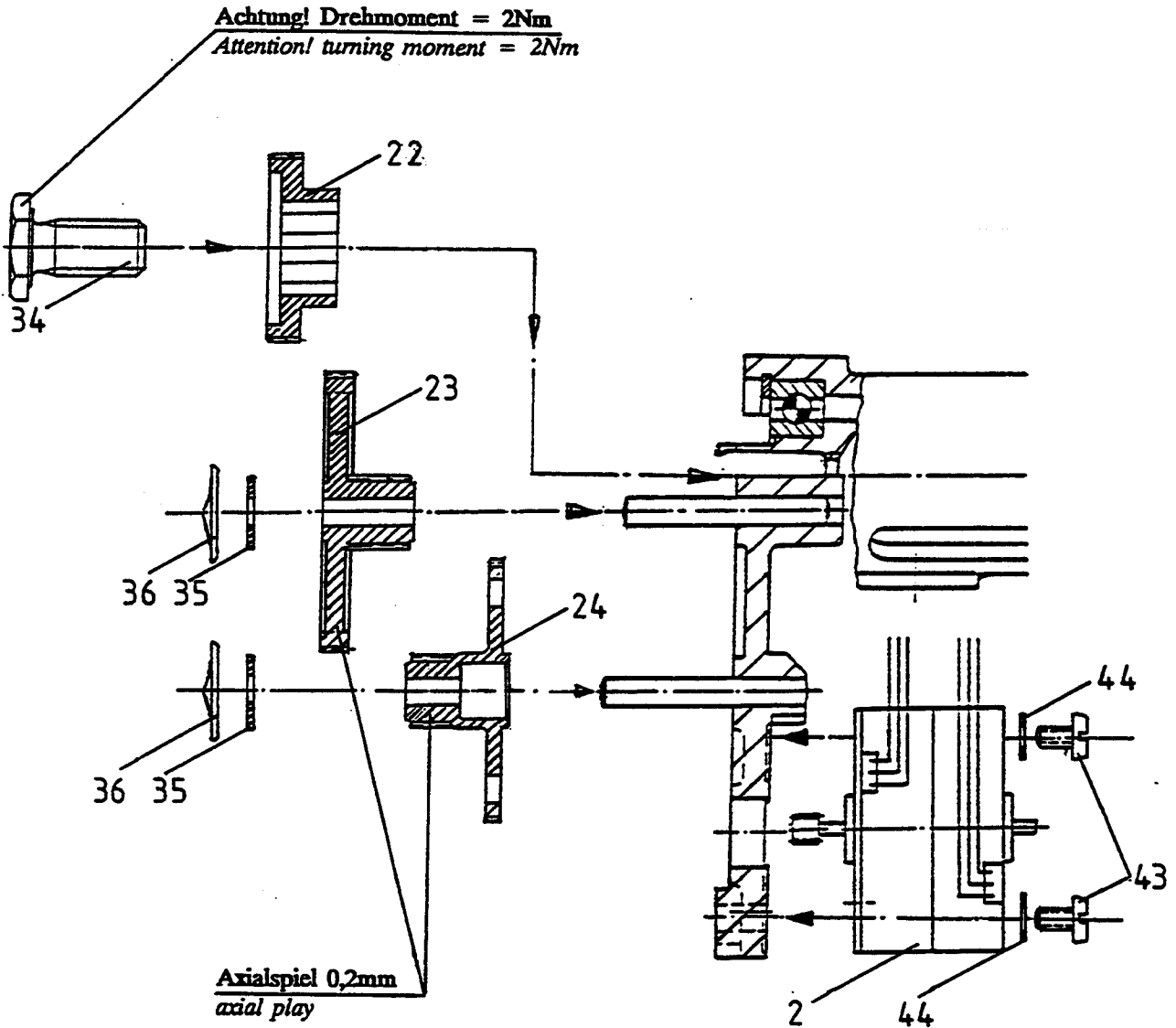
ATTENTION

Always use new upper part and lower part of syringe support when carrying out repairs!

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Repair instruction drive unit "subdomain B"

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Power pack rack 200V-230V-240V

M 513 000 000 N 04
Order No.: 34 50 310/2

consisting of:
1 piece Power pack rack
1 piece Fuse T80 mA
3 pieces Sealing caps for fuse holder
1 piece Type plate

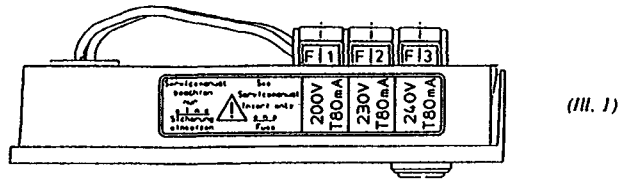
Power pack rack 100V-110V-120V

M 512 000 000 N 04
Order No.: 34 50 311/0

consisting of:
1 piece Power pack rack
1 piece Fuse T125 mA
3 pieces Sealing caps for fuse holder
1 piece Type plate

Commissioning of the power pack rack:

I. 200V - 230V - 240V

- Put fuse T80 mA in fuse holder, which corresponds to the desired mains input voltage.
in F1 - at 200V
in F2 - at 230V
in F3 - at 240V

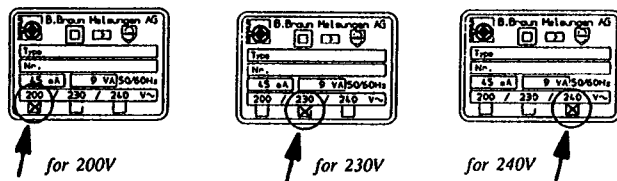
- Connect all 3 fuse holder with the included sealing caps

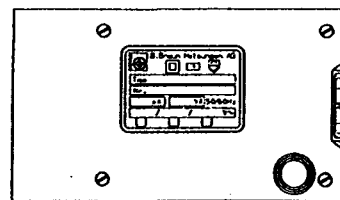
Loosen sealing caps
Turn approx. 45° counterclockwise with screw driver, lift caps with a small screw driver.

- Fit type plate at the bottom panel of the power pack rack

Type plate

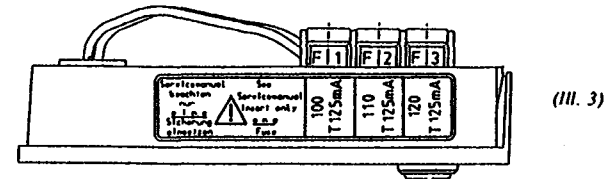
(III. 2)

- After fitting the type plate at the bottom panel, mark the adjusted input voltage with a marking tool at the position marked with an arrow.

II. 100V - 110V - 120V

- Put fuse T125 mA in fuse holder, which corresponds to the desired mains input voltage.
in F1 - at 100V
in F2 - at 110V
in F3 - at 120V

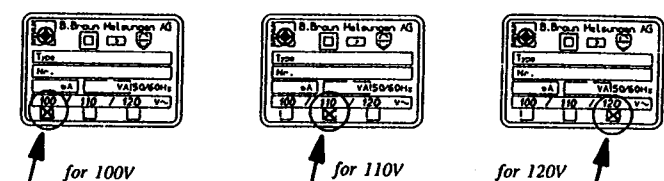
- Connect all 3 fuse holder with the included sealing caps

Loosen sealing caps
Turn approx. 45° counterclockwise with screw driver, lift caps with a small screw driver.

- Fit type plate at the bottom panel of the power pack rack

Type plate

Commissioning: Power pack rack PERFUSOR® secura FT

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A

INSPECTION WITH SERVICE

Manufacturer's recommendation for the PERFUSOR FT infusion syringe pump
Operator:

Fact.-No.: Appliance No.:
Year of purchase:

The INSPECTION with SERVICE is to be carried out according to the check list due reference to the service manual and the operation instructions subject to technical amendments and should be carried out within a period of 24 months. The used power pack have to be included in the inspection.

CHECK LIST

1	Record service data	12	Function ACCUMULATOR OPERATION
2	Unit + Power pack Cleanliness, completeness, damage Moisture attack and other influences ≠ Clean, seal	13	Switch on unit (mains supply) - Alarm tone - Display (mains, operation, alarm) - LCD Illumination
3	Open up unit - Visible inspection inside, where necessary clean - Check conformity with service- manual, modifications, etc. - Integrated power pack	14	Keyboard test/Display test *
		15	Prealarm and limit switch with signal at delivery rate 99.9ml/h *
4	Switch cover Rear wall ≠ where necessary replace	16	Force between the syringe supports - Limit at 50-70 N via spring path of 0.5-1mm if necessary take measurement through syringe pressure 0.8-1.2bar taking into consideration pressure loss *
5	Set up unit ready for use Seal		
6	Housing leakage current < = 0.01mA *		Peculiarities/remarks e.g.: exceptional findings - Specifications, Accessories - if necessary pump holder, firm standing, setting-up at the location, ambient conditions, interface operation, Accumulator operation and potential compensation - Manufacturer's data, modifications - Unit book - Last repairs, if necessary main changes to above mentioned established facts - Dangerous deficiencies - Directive safety - Repairs under Item R to be carried out/ carried out - To Point (X) of check list (e.g. description of faults)
7	Put into operation as per operating instructions - Delivery rate accuracy ±2% at max. rate		
8	Staff call and external power supply (plug connection and possible cable) FUNCTION *		
9	Easy spindle movement		
10	FUNCTION: ROTARY KNOB RECOGNITION FUNCTION: PRESSURE RECOGNITION FUNCTION: SYRINGE RECOGNITION (by fitting the appropriate syringe) *		
11	Type shield, Labels ≠ if necessary renew		
Documentation apart from Check list (i.e. unit book/customer service report) The INSPECTION with SERVICE was carried out by Date/Signature The order for this service was placed on		The next inspection date is scheduled for the Unit handed over to B.Braun Melsungen AG	
* Instructions for carrying out inspections on the following pages			

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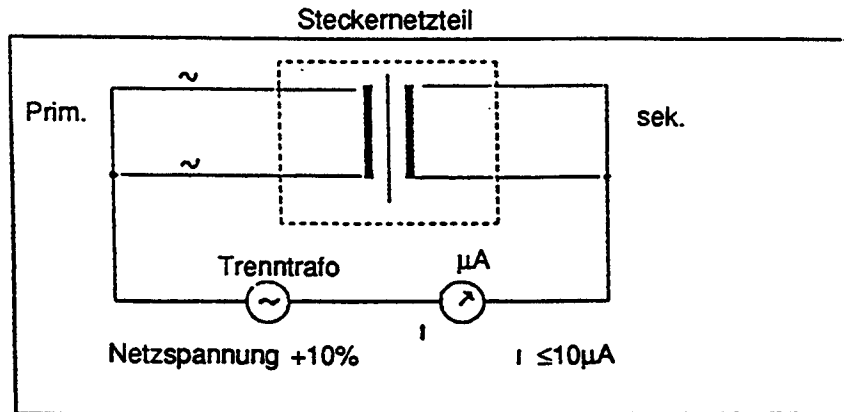
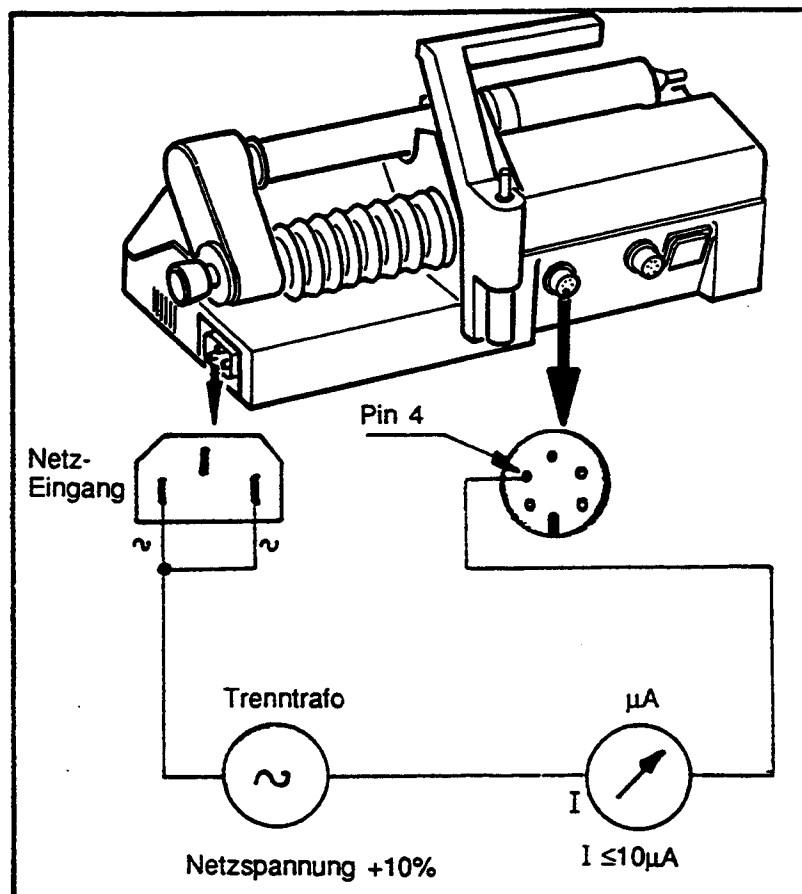
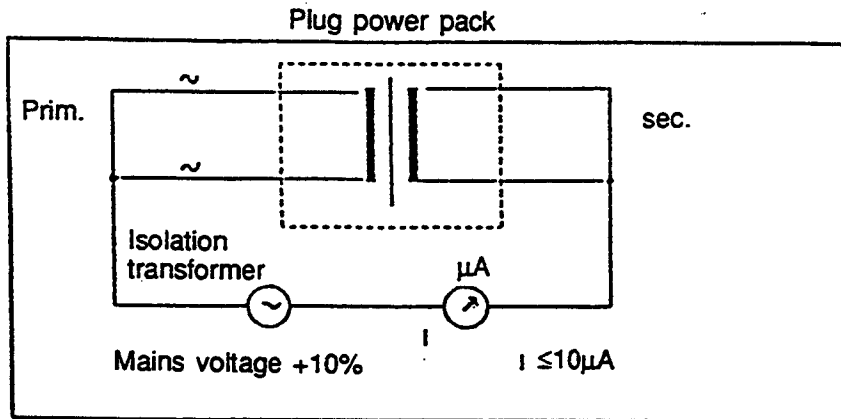
Instructions for carrying out inspection with service

On point 6:

Housing leakage current: The measurements are to be carried out in accordance with DIN IEC 601, VDE 0750, Part 1105.82, (Section 19.4 g) 5 and must be carried out in conjunction with the pertinent power pack.

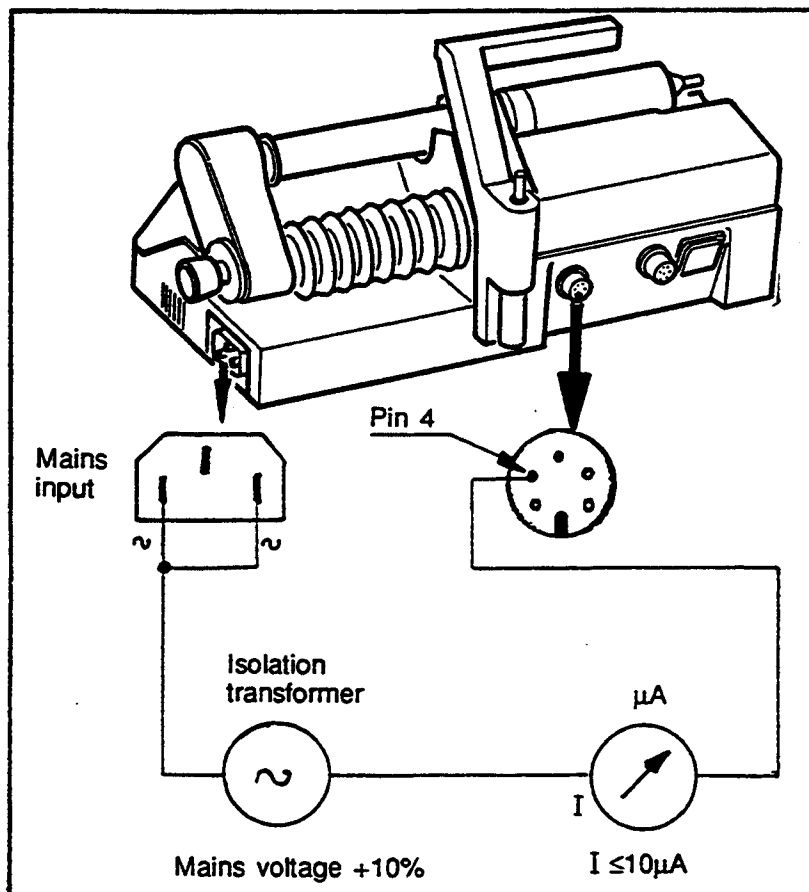
When using the plug power pack, the pertinent plug power pack can also be measured on its own.

The leakage current between primary and secondary side of the plug power pack corresponds with the maximum possible leakage current.

When using the integrated power pack:

The leakage current between primary and secondary side of the power pack corresponds with the maximum possible leakage current.

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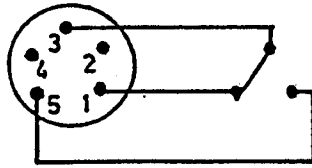
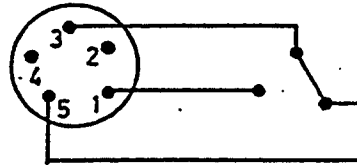
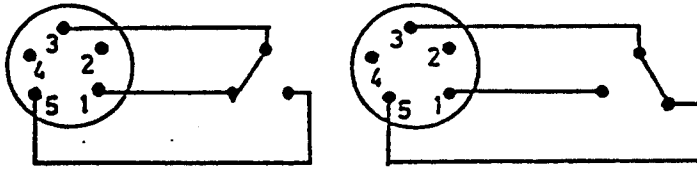
On point 10:

Rotary knob identification:

Start unit, release drive - working alarm

Pushbutton identification:

Start unit, push pressure pick-up towards the inside

- working alarm

Syringe identification:

Start unit without syringe

- working alarm

to point 14:

Keyboard test

a. Plug-in test adapter

Test step	Display
Switch on	A04
Key "."	00.0 ml/h
Key 1	00.1 ml/h
Key 2	00.2 ml/h
Key 3	00.3 ml/h
Key 0	0.00 ml/h
Key 4	00.4 ml/h
Key 5	00.5 ml/h
Key 6	00.6 ml/h
Key 7	00.7 ml/h
Key 8	00.8 ml/h
Key 9	00.9 ml/h
Key C	00.0 ml/h
Key CC	"CC" No. = 1...99
Key ml	00 0 ml
Key Start/Stop	Running time -h

Disconnect test adapter

"Start/Stop"	Beep	4 : 1 sec
"Alarm suppression"	Beep	off

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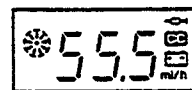
to point 15: *Prealarm: start unit with a syringe filled with approx. 7ml and at a delivery rate of 99 ml/h. Prealarm is signalled after approx. 2 Min. The final alarm is signalled after a further 3 Min. Start unit once again. Final alarm is released again after a few seconds.*

to point 16: *Force between the syringe supports:
Suggested measurement for syringe pressure.*

Attention! use new syringe for each measurement.

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The spare parts marked with ⊕ can only be ordered by request and are deliverable in larger amounts.

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This maintenance reference must not be copied, partly or whole, nor made accessible to third parties without our consent, and remains the property of our company.

Service-Manual

Perf.sec.FT [91]

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03/91

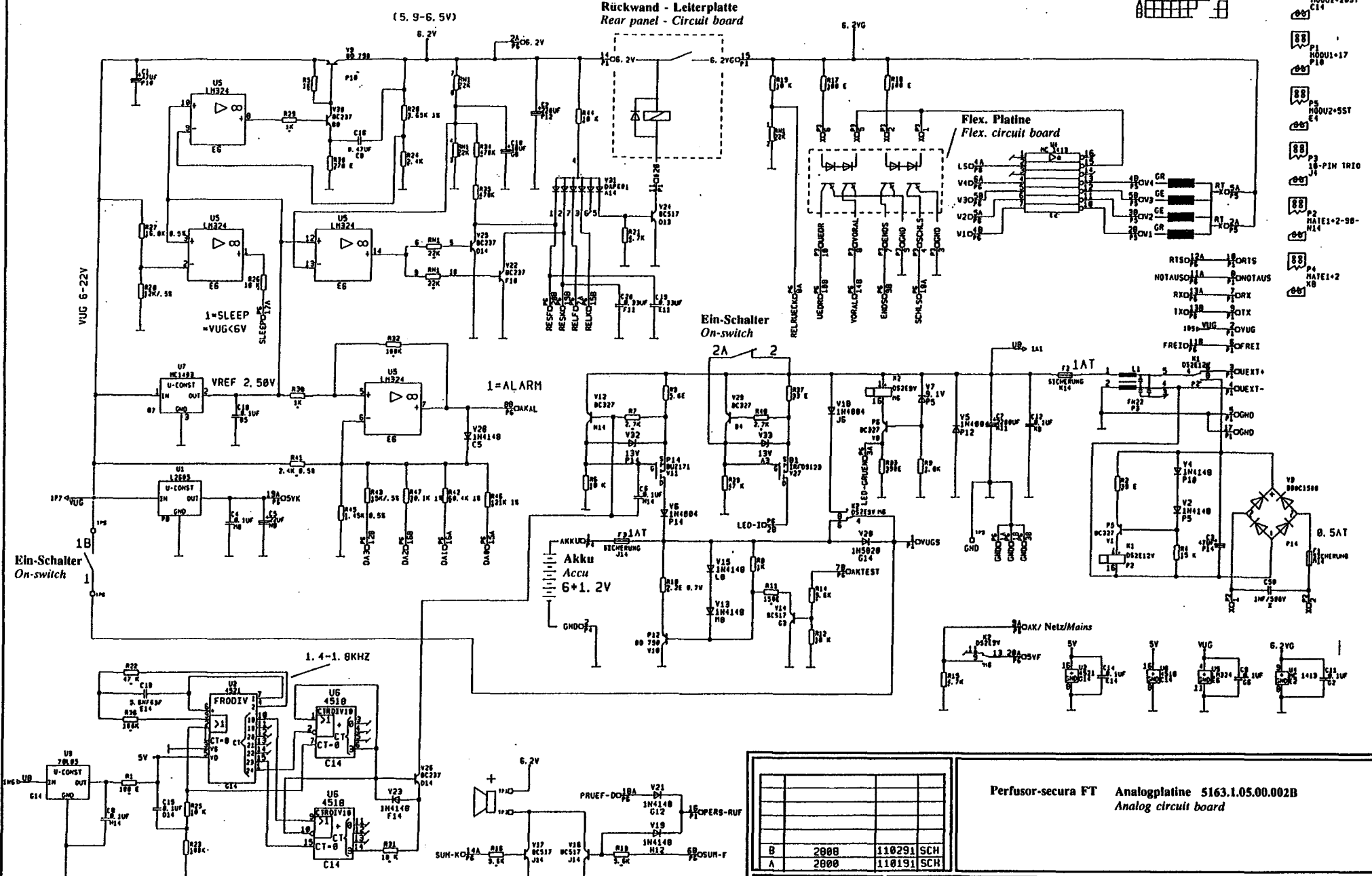
B. Braun Melsungen AG

TECHNmed
Service

A

Rückwand - Leiterplatte
Rear panel - Circuit board

- 88 P5 HO0U2+20ST C14
- 88 P1 HO0U1-17 P18
- 88 P5 HO0U2+5ST C4
- 88 P3 18-PIN TRIO J4
- 88 P2 HATE1-2-90-N14
- 88 P4 HATE1-2 NB

- RISOU2A 18 CRTS
- NOTAUSOU2A 18 CHOTAUS
- RXC13A 18 CRK
- TXC13B 18 CTX
- 108-VUG 18 OVUG
- FRIC13B 18 CFREI
- CUEXT+ 18
- CUEXT- 18
- OGND 18
- OGND 18
- 0.5AT 18
- SICHERUNG 18

Perfusor-secura FT Analogplatine 5163.1.05.00.002B
Analog circuit board

B	2808	110291	SCH
A	2808	110191	SCH

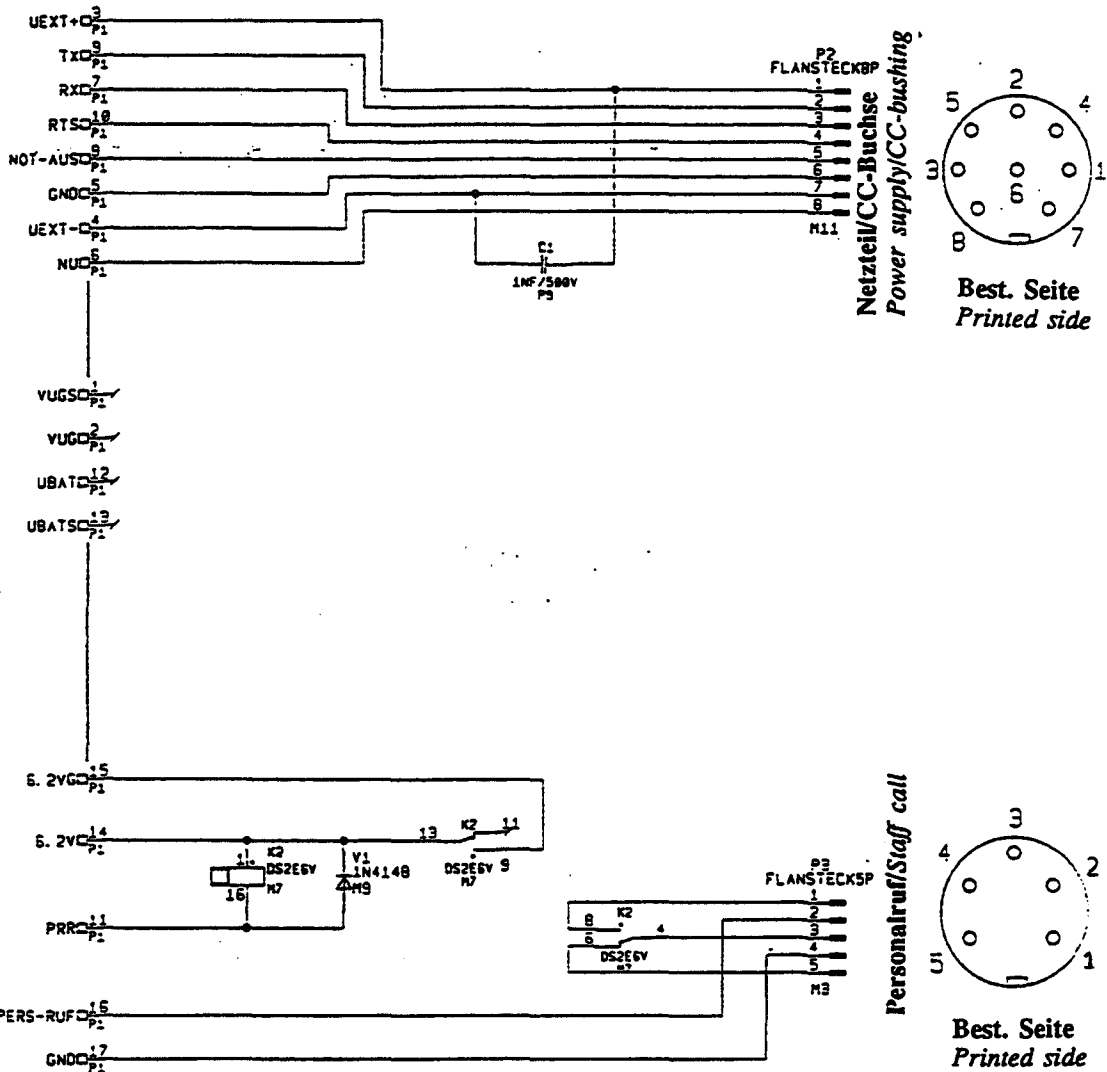
These Service-Unterlage Daten to assist our customers with installation and use. They are not intended to be used as a substitute for the original product and do not constitute a warranty. The manufacturer is not responsible for any damage or loss of data, but we will accept liability for the parts without our consent, as long as they are the property of our company.

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of B. Braun Medical AG. All rights reserved. No part