


POCT

Point of Care Testing
(Dry Fluorescence Immunoassay)



“ **Quantitative, Rapid and High Sensitive
Reliable Result (QC system, QR code calibration)** ”

► **Time-resolved Fluorescence Immunoassay (TRFIA) Method:**

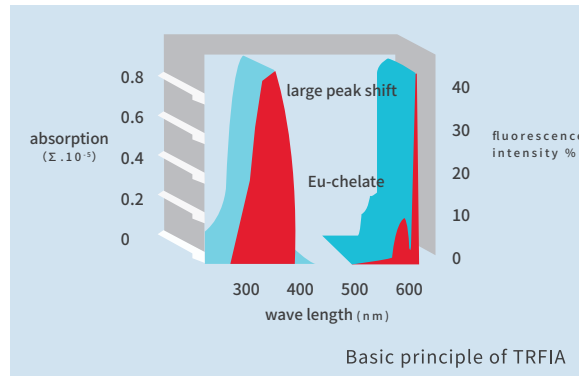


TRFIA is super-sensitive detection technique characterized by specific fluorescence of rare earth ions. It is not only highly sensitive, but also overcomes the instability of enzyme marker and is the best choice for immunological detection. The high fluorescence intensity and long life of labeled ionic chelates are beneficial to eliminate the influence of fluorescent substances in samples and environment on the test results.

► **Analyzer Introduction:**

Analyzer uses the advanced method of Time-resolved Fluorescence Immunoassay (TRFIA), for the in-vitro quantitative detection of bio-markers for Diabetes Mellitus, Inflammation, Cardiovascular Diseases, Hormone, Gastric Diseases, Renal Diseases, Tumor, etc.

Application: Laboratory, ER, Cardiology, ICU, Respiratory, Pediatrics, etc.

► **Application:**

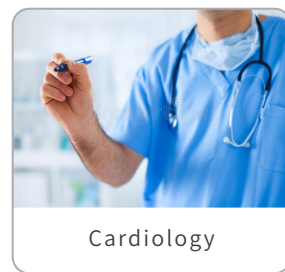

LS-1100




Dry Fluorescence Immunoassay Analyzer (Portable)



► Features:

Method	Time-resolved Fluorescence Immunoassay (TRFIA)
Specimen	Serum/Plasma/Whole Blood/Urine
Weight	1.3kg
Dimensions	225mm×152mm×105mm (L×W×H)
Screen	7 inch touch screen
Data Storage	10000
Printer	Built-in thermal printer
Battery	Built-in lithium battery (standby time: 10hrs)
Communication	RS232(LIS/HIS), RJ45, USB, WIFI, Bluetooth


► Easy Operation:

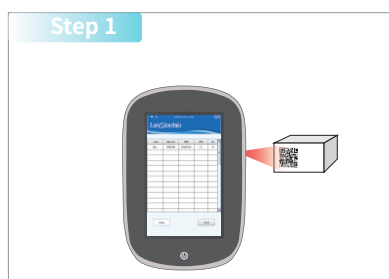

QR Code Calibration



Information Input

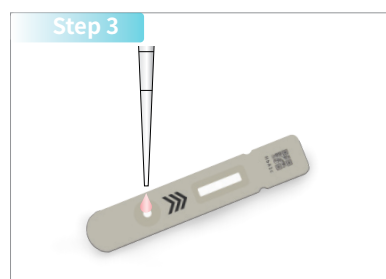

Sample Dispense



Automatic Detection



Result Output

LS-2100

LanSionbio®

Dry Fluorescence Immunoassay Analyzer (High Throughput)

- **High Sensitive**
- **Reliable Results**
(QC system+QR Code Calibration+Inner temperature control)
- **High Throughput**
(8 channels)
- **Rapid**
(in 10 secs)



► Features:

Method	Time-resolved Fluorescence Immunoassay (TRFIA)
Specimen	Serum/Plasma/Whole Blood/Urine
Weight	7.5kg
Dimensions	390mm × 290mm × 205mm (L×W×H)
Screen	10.1 inch HD touch screen
System	Android
Test Speed	120T/H
Data Storage	10000
Printer	Built-in thermal printer
Communication	RS232(LIS/HIS), RJ45, USB, WIFI, Bluetooth


► Easy Operation:




QR Code Calibration



Information Input

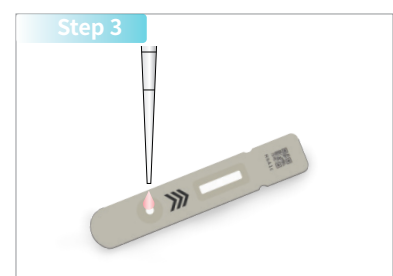


Sample Dispense

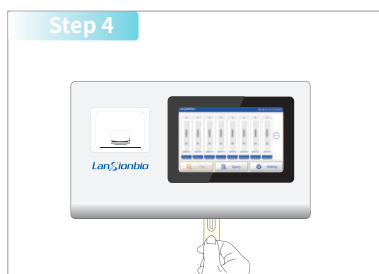

Insert Strip



Automatic Detection
(Up to 8 samples being detected simultaneously)

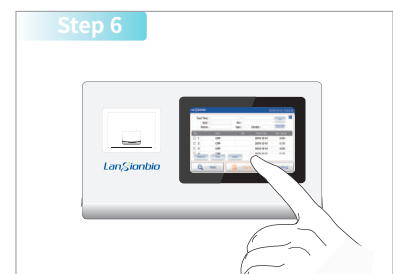

Result Output

—POCT One-stop Solution—

LS-4000

Lansionbio®

Dry Fluorescence Immunoassay Analyzer (Handheld)



Quantitative

Reliable Result

High sensitive

► Features:

Method	Time-resolved Fluorescence Immunoassay (TRFIA)
Specimen	Serum/Plasma/Whole Blood/Urine
Weight	600g
Dimensions	191mm×84.5mm×45mm (L×W×H)
Screen	5.0 inch touch screen
Data Storage	50000
Battery	Built-in lithium battery (super standby time)
Communication	RS232, Type-C USB, WIFI, Bluetooth and 4G

Lansion Biotechnology Co., Ltd.

► **Test Items:**

Category	Test Item	Specimen Type	Sample Volume	Reaction Time	Measuring Range
COVID-19	Antigen	Nasal Swab	100 μ l	15min	-
	IgM/IgG	S/P	5 μ l	15min	-
	NAb	S/P	5 μ l	3min	-
Diabetes	HbA1c	WB	5 μ l	15min	3-14%
Inflammation	CRP	S/PWB	5 μ l	3min	0.5-200 μ g/mL
	PCT	S/P	100 μ l	10min	0.1-50ng/mL
	SAA	S/P/WB	5 μ l	3min	2-300 μ g/mL
	SAA/CRP	S/P/WB	5 μ l	5min	SAA:2-300 μ g/mL
					CRP:0.5-200 μ g/mL
	PCT/CRP	S/P	5 μ l	3min	PCT:0.1-50ng/mL
					CRP:0.5-200 μ g/mL
	IL-6	S	100 μ l	15min	5-2000pg/mL
IgG4	S/P	100 μ l	15min	0.055-3.3g/L	
Cardiac	cTnI	S/P	100 μ l	10min	0.05-40ng/mL
	CK-MB	S/P	100 μ l	10min	0.2-80ng/mL
	Myo	S/P	100 μ l	10min	20-500ng/mL
	cTnI/CK-MB/Myo	S	100 μ l	10min	cTnI:0.05-40ng/mL
					CK-MB:0.2-80ng/mL
					Myo:20-500ng/mL
	NT-proBNP	S/P	100 μ l	15min	50-25000pg/mL
	D-Dimer	P	100 μ l	10min	0.1-10 μ g/mL
	H-FABP	S/P	100 μ l	5min	1ng/mL-200ng/mL
	BNP	S/P	100 μ l	15min	5-5000pg/mL
Lp-PLA2	S/P	100 μ l	10min	10-1000ng/mL	
Hormone	TSH	S/P	100 μ l	15min	0.1-60 μ IU/mL
	TT3	S/P	100 μ l	15min	0.5-10nmol/mL
	TT4	S/P	100 μ l	10min	10-350nmol/mL
	25-OH-VD3	S/P	100 μ l	10min	11ng/mL-43ng/mL
	25-OH-VD	S/P/WB	S/P:5 μ l	10min	5.0-70ng/mL
			WB:20 μ l		
	β -HCG	S/P	50 μ l	15min	2.0-20000mIU/mL
	AMH	S	100 μ l	10min	0.1-50ng/mL
	LH	S	100 μ l	10min	1.02-200mIU/mL
	FSH	S	100 μ l	10min	1-200mIU/mL
	PRL	S	100 μ l	15min	200-4000 μ IU/mL
Progesterone	S/P	100 μ l	10min	0.8-50ng/mL	
Gastric Function	PG I /PG II	S	100 μ l	10min	PGI: 10-160 ng/mL
					PGII: 6.25-100 ng/mL
Tumor	PSA	S/P	100 μ l	10min	0.1-100ng/mL

New items are available soon!

Lansion Biotechnology Co., Ltd.

Add: No.2 Qiande Road, Jiangning District, Nanjing, China

E-mail: biz@lansionbio.com

Web: en.lansionbio.com

Tel: +86-25-5857 7600

Fax: +86-25-5875 8600

