



<Marks on the Package>

Marks	Location	Meanings
	Package box	Lot No.
	Package box	Not sterilized

<Conventions Used in This Manual>

This manual uses the following conventions to make it easy to understand operations.

Convention	Meaning
CAUTION	Explains situations that may cause damage to equipment if not avoided.

2. Method of Use

CAUTION	Use this product only in combination with the our endoscope. There is a risk of damaging the equipment.
----------------	---

<Operation Procedure>

For details on how to use this product, refer to the Operation Manual (Cleaning, Disinfection and Storage) of the endoscope in use.

3. Main Specification

<Operating Environment>

Temperature : +10 to +40°C
 Relative humidity : 30 to 85% (except when there is condensation)
 Atmospheric pressure : 70 to 106 kPa (within range of atmospheric pressure)

<Storage Environment>

CAUTION	Do not store this product in a place which does not meet the storage environment. There is a risk of failure.
----------------	---

Temperature : +10 to +40°C
 Relative humidity : 30 to 85% (except when there is condensation)
 Atmospheric pressure : 70 to 106 kPa (within range of atmospheric pressure)

<Applicable Endoscope>

This product is used in combination with our endoscopes. The applicable endoscopes are listed in the following table. For the endoscopes other than listed here, please consult your local dealer.

The list of applicable endoscopes (example)

200 series	400 series	500 series	Other
EC-201WI	EC-450B15	EC-530FI	FC-1Z
EC-201WL	EC-450LP5	EC-530FL	FG-1Z
EC-201WM	EC-450WL5	EC-530FM	FG-1ZP
EC-250LP5	EC-450WM5	EC-530IT	
EC-250WI5-H-	ED-450XL5	EC-530LP	
EC-250WI5-S-	EG-450PE5	EC-530LT	
EC-250WL5-H-	EN-450P5/20	EC-530MP-H-	
EC-250WL5-S-	EN-450T5	EC-530MP-S-	
EC-250WM5	EN-450T5/W	EC-530MT	
EG-201FP		EC-530WI-H-	
EG-250CT5		EC-530WI-S-	
EG-250PE5		EC-530WL	
EG-250WR5		EC-530WM-H-	
EG-270N5		EC-530WM-S-	
		EC-590WI	
		EC-590WL	
		EC-590WM	
		EC-590ZW/L	
		EC-590ZW/M-H-	
		ED-530XT	
		ED-530XT8	
		EG-530CT	
		EG-530FP	
		EG-530N	
		EG-530NP	
		EG-530WR	
		EG-590WR	
		EG-590ZW	
		ES-530WE	

OPERATION MANUAL



202B1261806A
 110912-2.0-DT-E2
 Printed in Japan

FW536C

1. Important Safety Information

For the USA Market - CAUTION:

Federal law restricts this device to sale by or on the order of a physician.

<Intended Use>

This product is the cleaning adapter used in combination with our endoscopes in medical facilities.

Do not use this product for any other purpose.

<Precautions in Using This Product>

Before using this product, carefully read the Operation Manual (Cleaning, Disinfection and Storage) of the endoscope in use.

<Sterilization>

This product has not been sterilized.

When using this product, perform disinfection or sterilization at the level appropriate to the intended use, in accordance with the Operation Manual (Cleaning, Disinfection and Storage) of the endoscope in use.

<Preparation and inspection before use>

Prior to using this product, prepare a spare one to avoid unexpected accidents such as equipment failure. If a replacement is not available, you may not be able to continue endoscopic procedures.

Make sure to check this product before use according to the procedures provided in the Operation Manual (Cleaning, Disinfection and Storage) of the endoscope in use, to avoid unexpected accidents and take full advantage of this product's capabilities. If the inspection result shows any defects, do not use this product and consult your local dealer.

<Applicable Endoscope>

This product is used in combination with our endoscopes described in "3. Main Specification." Do not use this product in other combinations.

<Disposal>

When disposing of this product, comply with local laws and regulations in your area. Determine whether or not this product is to be treated as infective waste, depending on the use.

<Medical Device Directive>

This product complies with the requirements of
European Directive 93/42/EEC.
Classification : Class IIa

CE 0123



FUJIFILM Corporation

26-30, Nishiazabu 2-chome, Minato-ku, Tokyo 106-8620, Japan



FUJIFILM Europe GmbH

Heesenstrasse 31, 40549 Duesseldorf, Germany