

## Oxylog<sup>®</sup> 1000 Emergency & Transport Ventilation

The Oxylog<sup>®</sup> has been the natural choice of emergency care ventilator for more than 25 years. The Oxylog<sup>®</sup> 1000 is the most compact ventilator in the Oxylog<sup>®</sup> range.



## Benefits

---

### A system for a safer work environment

The Oxylog 1000 offers first aid ventilation of patients in emergency situations. Designed to be used outdoors, its intuitive operation, robustness and transportability make the Oxylog 1000 complete in its class. The ventilator has integrated audible and visual alarms that monitor both the airway pressure and supply pressure to aid in patient safety.

All functions are pneumatically operated, so the ventilator does not depend on any electrical power source. Thanks to its fundamental design, the Oxylog 1000 also stands for great reliability, ready to meet rough conditions during your mission.

## Accessories

---



MT7494\_2004

### Caddy & Compact Caddy

Order Number 57 03 300



D-15194-2010

### Carrying System 1000

Order Number 2M86001

## Accessories



D-14931-2010

### Allround wallholder

Order Number 57 04 216



D-61659-2012

### Alduk IV O<sub>2</sub> Compact Pressure Reducer

Order Number 57 04 500

## Related Products



D-9219-2009

### Dräger Oxylog® 3000 plus

Offering high ventilation performance with features such as AutoFlow® integrated capnography and non-invasive Ventilation, the compact and robust Oxylog® 3000 plus helps you transport your patients safely and provides feedback on correctness of intubation and ventilation effectiveness. The Oxylog® 3000 plus gives you confidence to master even the most demanding situations.

## Related Products



---

### Oxylog® 2000 plus

Step up your performance with Oxylog® 2000 plus. The Oxylog® 2000 plus supports you in your daily challenge of saving peoples lives, no matter where the call takes you. Invasive or non-invasive, Oxylog® 2000 plus can meet this challenge by putting essential ventilation tools at your fingertips. The Oxylog® 2000 plus can make all the difference.

## Technical Data

Oxylog® 1000 – a time-cycled, volume controlled and pressure limited emergency ventilator for the controlled ventilation of patients who require a minute volume of at least 3 L/min.

Dimensions (W x H x D)	215 x 90 x 215 mm / 8.5 x 3.5 x 8.5 inches (excl. handle)
Weight	3.15 kg / 7.3 lbs

### Drive Gas

Medical grade O <sub>2</sub> or in exceptional cases compressed air	
Supply pressure	2.7 to 6 bar / 40 to 88 psi at 60 L/min

### Performance Data

Ventilation mode	IPPV/ CMV
PEEP ventilation	with optional PEEP valve
Principle of operation	Flow chopper
Control	Time-cycled, volume-constant
Ventilation frequency, smoothly variable	4 to 54 1/min
Minute volume, smoothly variable	3 to 20 L/min
I:E ratio (fixed)	1 :1.5
Max.airway pressure (Pmax), smoothly variable	25 to 60 mbar / cm H <sub>2</sub> O

### O<sub>2</sub> concentration of Ventilation Gas when O<sub>2</sub> driven

Switch to "Air Mix"	approx 60% by vol. O <sub>2</sub>
Switch to "No Air Mix"	100% by vol. O <sub>2</sub>
Gas consumption of control	approx 1.0 L/min
Dead space volume	approx 12 mL
Device compliance	approx. 1 mL/mbar / cm H <sub>2</sub> O
Safety valve opening pressure	80 mbar / cm H <sub>2</sub> O
Pressure gauge display	-10 to +80 mbar / cm H <sub>2</sub> O

### Alarm Functions

Supply pressure low (Psupply)	Supply pressure drops below 2.7 bar / 40 psi
Airway pressure high (Paw high)	Actual value exceeds set value (Pmax)
Airway pressure low (Paw low)	A pressure of 10 mbar/cm H <sub>2</sub> O is not exceeded during inspiration
The alarms are both visual and audible.	
They are provided by purely pneumatic components and do not require any power supply.	

### Conditions for Operation

Temperature range	-18 °C to +50 °C / 0 to 122 °F
Relative humidity	15% to 95% rel humidity
Ambient pressure	700 to 1100 h Pa
Vibration tested	in acc.with MIL STD 810 F, methode 514.5
Airworthiness	in acc.with RTCA DO-160 D, section 8
Classification acc. to EC Directive 93/42/EEC	Class IIb
UMDNS Code	18-098

Typical operating time MV = 10 L/min

- 2.5 L cylinder / 200 bar: approx. 90 min for "Air Mix", approx. 45 min for "No Air Mix"
- E-type O<sub>2</sub> cylinder: approx. 112 min for "Air Mix", approx. 56 min for "No Air Mix"
- D-type O<sub>2</sub> cylinder: approx. 64 min for "Air Mix", approx. 32 min for "No Air Mix"

## Notes

### CORPORATE HEADQUARTERS

Drägerwerk AG & Co. KGaA  
Moislinger Allee 53–55  
23558 Lübeck, Germany  
[www.draeger.com](http://www.draeger.com)

### Manufacturer:

Drägerwerk AG & Co. KGaA  
Moislinger Allee 53-55  
23558 Lübeck, Germany

### REGION DACH

Drägerwerk AG & Co. KGaA  
Moislinger Allee 53–55  
23558 Lübeck, Germany  
Tel +49 451 882 0  
Fax +49 451 882 2080  
[info@draeger.com](mailto:info@draeger.com)

### REGION EUROPE

Drägerwerk AG & Co. KGaA  
Moislinger Allee 53–55  
23558 Lübeck, Germany  
Tel +49 451 882 0  
Fax +49 451 882 2080  
[info@draeger.com](mailto:info@draeger.com)

### REGION MIDDLE EAST, AFRICA

Drägerwerk AG & Co. KGaA  
Branch Office  
P.O. Box 505108  
Dubai, United Arab Emirates  
Tel +971 4 4294 600  
Fax +971 4 4294 699  
[contactuae@draeger.com](mailto:contactuae@draeger.com)

### REGION ASIA / PACIFIC

Draeger Medical  
South East Asia Pte Ltd.  
25 International Business Park  
#04-27/29 German Centre  
Singapore 609916, Singapore  
Tel +65 6572 4388  
Fax +65 6572 4399  
[asia.pacific@draeger.com](mailto:asia.pacific@draeger.com)

### REGION CENTRAL AND SOUTH AMERICA

Dräger Panama Comercial  
S. de R.L.  
Complejo Business Park,  
V tower, 10th floor  
Panama City  
Tel +507 377 9100  
Fax +507 377 9130  
[contactcsa@draeger.com](mailto:contactcsa@draeger.com)

Locate your Regional Sales  
Representative at:  
[www.draeger.com/contact](http://www.draeger.com/contact)

